

DROP-IT[®]

Quick Release Needle Holder

May be used with standard blood collection devices.

Suggested use with the...

Punctur-Guard[®] Blood Collection Needle

- Internal blunting technology which is integral to the needle itself.
- Blunts while in the patient's vein, eliminating exposure during those first few critical moments.
- Reduced needle sticks by 76%, the highest rate of any device studied.

*Morbidity and Mortality Weekly, 1997; 46: 21-25.



Sharp Entry...



Blunt Exit

Recommended for
up to 100 uses!

Push button to
release needle



NOTE: During use, exercise care not to depress needle release button when threading needle onto holder or during procedure.

Always inspect the holder for integrity and cleanliness before each use.

To Disinfect: Soak in 10% bleach solution for 10-20 minutes, then rinse thoroughly. Wash with detergent, rinse and dry. If holder is damaged, discolored or remains soiled after disinfection, discard. **Maximum 20 cleanings per holder. DO NOT soak overnight.**



GAVEN MEDICAL, LLC

369 Lang Boulevard
Grand Island, NY USA 14072
1-800-647-6474
www.gavenmedical.com